

## PROPANO AF

Ready to use alcoholic spray-disinfectant - aldehyde-free

**Product information:** PROPANO AF is a disinfectant for alcohol resistant surfaces, equipment and furniture (for example: dentist's chairs, work spaces in laboratories and practices, tool holders and angles, inventory in operating and sanitary rooms). For the application in hospitals, health services, residential homes and laboratories. Wet the whole object to be disinfected from about 30 cm distance. The surface dries rapidly and residue-free without wiping.

- aldehyde-free
- quats-free, no respiratory-related irritation
- excellent material compatibility and material preserving
- available perfumed as *Classic* and *Lemon* as well as *Neutral* (perfume free)

**Active ingredients:** 100 g containing:  
 ethanol 40,50 g  
 n-propanol 9,20 g

**Microbiological activity and acting time:**

Spectrum	Organic load		Acting time
	without	with	
Bacteria <sup>1)</sup>	x	x	30 seconds
Tuberculosis <sup>2)</sup>	x		
		x	1 minute
Viruses <sup>3)</sup>	HCV/ HBV/ HIV	x	30 seconds
	Adenovirus	x	
	Rotavirus	x	
	Vacciniavirus	x	
	SARS	x	
Fungus <sup>1)</sup>	x	x	

<sup>1)</sup> Surface disinfection according to DGHM

<sup>2)</sup> *M. terrae*

<sup>3)</sup> Virucidal against enveloped viruses according to RKI recommendation Bundesgesundheitsblatt 01-2004

**Packing unit:** 125 ml bottle  
 250/ 500/ 1000 ml spray bottle  
 5/ 10 liter canister

**Remarks:** Expert Reports:  
 Prof. Dr. H.-P. Werner, Schwerin; Prof. Dr. R. Schubert, Frankfurt/ M.; Dr. J. Steinmann, Bremen; Prof. Dr. A. Kramer, Greifswald.

- Contains alcohol. Check for material compatibility. Do not use with acrylic glass.
- Labelled according to Gefahrstoffverordnung (Ordinance on Hazardous Substances): R11: Highly flammable. 27 °C Flash point according to DIN 51755.
- Do not swallow; do not apply close to eyes.
- Keep away from children.
- Keep away from fire.

DGHM listed.

CE 0124